

TECHNICAL DATA SHEET

BIOSECURITY CALF SCOUR TEST KIT


An ideal aid in the rapid detection of rotavirus, coronavirus, *E. coli* and *cryptosporidium parvum* in calves.

PRODUCT BENEFITS:

- No need to wait for lab results, just test and go in 10 minutes!
- Detects E. coli F5 (ETEC) (K99), rotavirus type A, coronavirus (BCoV) and cryptosporidium parvum
- Each kit contains 5 calf tests.
- Simple visual confirmation.
- Highly sensitive and reliable.

INSTRUCTIONS FOR USE:

- 1. Take the faeces directly from the rectum of the calf. If the samples are liquid, take a spoonful.
- 2. If faeces are solid, remove the excess amount using a spatula.
- 3. Dilute the spoon content in the liquid of the small tube called sample tube.
- 4. Shake the sample tube to homogenize.
- 5. Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.
- 6. Remove the tube's protective caps.
- 7. Break the blue end.
- 8. Place the cartridge on a completely level surface. Deposit three drops in each small hole. Leave the device on a flat surface and wait for 10 minutes. The liquid contained in the sample tube moves to the strip and slowly migrates along the strips.
- 9. After 10 minutes, read the results using the picture below as standard.

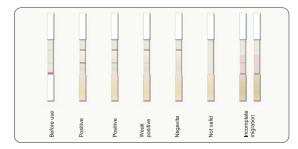

If test is positive, seek veterinary advice immediately

Shelf life:

1 year from date of manufacture.

Storage:

Store between 0°C to 25°C. Keep out of direct sunlight.

For health and safety information, please refer to the Safety Data Sheet.

PROGLEDE

CALF SCOUR

TEST HIT

TEST

SIZES AVAILABLE:

1 kit (5 calf tests) - Code P829